




For the explantation from our screws will need in the normal case only a screw driver. This has a standardised hexagonal connection. Please take from the table the respective size. Details to the screw drivers you will find on the end of the list

Wrench size Connection		Cross	Special	2.5	2.5	3.5
Cannulation in mm		-	-	1.5	1.8	2.2
						
Special foot screws						
TWISTER screws, D 2.0 mm, recessed head		✓				
TWISTER-I screw, D 2.0 mm			✓			
TWISTER-I screw, D 2.7 mm			✓			
Cannulated screws						
Cannulated screw D 3.0 mm, HS*				✓		
Cannulated screw D 4.0 mm, HS*					✓	
Cannulated screw D 7.0 mm, HS*						✓

HS* - Cannulated screw with hexagon socket




For the explantation from our screws will need in the normal case only a screw driver. This has a standardised hexagonal connection. Please take from the table the respective size. Details to the screw drivers you will find on the end of the list

Wrench size Connection		1.5	2.0	2.5	3.0	4.0
Cannulation in mm		1.0	1.1	1.8	1.8	2.7
						
Compression screws, cannulated						
CIRRUS® screw mini, D 2.3 mm, Compression screw, HS*		✓				
CIRRUS® screw, D 3.0 mm, Compression screw, HS*			✓			
EPIUS screw, D 4.0 mm, Compression screw, HS*				✓		
METIUS screw, D 4.5 mm, Compression screw, HS*				✓		
CLAVIUS screw, D 5.5 mm, Compression screw, HS*					✓	
CLAVIUS bolt, D 5.5 mm, Compression screw, HS*					✓	
CONTUS screw, D 7.5 mm, Compression screw, HS*						✓
CONTUS bolt, D 7.5 mm, Compression screw, HS*						✓

HS* - Cannulated screw with hexagon socket



HS Co	Code N°	Description
Cross	750.102038	 <p>Recessed head screw driver, for TWISTER screw D 2.0 mm, with handle and holding sleeve</p>
Special	750.102008	 <p>Screw driver, for TWISTER-I screws, with handle</p>
HS 1.5 ID 1.0	750.102063	 <p>Cannulated hexagon screw driver, for CIRRUS® screws mini, D 2.3 mm, for wires D 0.8 mm, with handle</p>
HS 2.0 ID 1.1	750.102027	 <p>Cannulated hexagon screw driver, for CIRRUS® screws D 3.0 mm, with measuring scale and handle</p>
HS 2.5 ID 1.5	750.102014	 <p>Cannulated hexagon screw driver, for screws D 3.0 mm, for wires D 1,4 mm, with handle</p>
HS 2.5 ID 1.8	750.102015	 <p>Cannulated hexagon screw driver, for screws D 4.0 mm, D 4.5 mm, for wires D 1.6 mm, with handle</p>
HS 3.0 ID 1.8	750.102056	 <p>Cannulated hexagon screw driver, for CLAVIUS screws D 5.5 mm, for wires D 1.6 mm, with handle</p>
HS 3.5 ID 2.3	750.102016	 <p>Cannulated hexagon screw driver, for cannulated screws D 7.0 mm, for wires D 2.0 mm, with handle</p>
HS 4.0 ID 2.7	750.102055	 <p>Cannulated hexagon screw driver, for CONTUS screws D 7.5 mm, for wires D 2.5 mm, with handle</p>

SWS = Wrench size | Co = Connection | HS = Hexagon socket | ID = Internal diameter



intercus

Instruments for explantation
Foot surgery - screws

For more informations please connect our responsible sales representatives or our back office:

Export

INTERCUS GmbH
Zu den Pfarreichen 5
07422 Bad Blankenburg
GERMANY

Phone: +49 36741 588-0
Fax: +49 36741 588-285
E-Mail: info@intercus.de



National: INTERCUS Vertriebs GmbH | In der Flecke 22-23 | 07422 Bad Blankenburg
Tel.: +49 36741 586265 | Fax: 586469 | E-Mail: info@intercus-vertrieb.de
International: INTERCUS GmbH | Zu den Pfarreichen 5 | 07422 Bad Blankenburg
Tel.: +49 36741 588-0 | Fax: 588-285 | E-Mail: info@intercus.de | GERMANY

www.intercus.de